



AT Ultra Light™ Computer Switch Interface Piano Mount User Guide

Introduction

The following information is provided in hopes to make connecting the Ultra Light™ Piano Interface to the CSI (Computer Switch Interface) even easier.

Directions

1. Plug the switches into the desired locations.
2. Plug the CSI into the computer
3. Set the switch actions to mouse control by pressing the CSI Advance button ➡➡ until the top light is lit.
You're ready to go!

There are many ways to configure the switches. Here is a popular one. Because, it allows one finger to move the mouse's many combinations:



Upper Right (Yellow + Blue)

Upper Left (Yellow + Orange)

Plus all the other operation directions and control click.



Marty

Assistive Technology Engineering and Consultation Inc. (Atec)

Engineering: (651) 628-0896

Fax: (612) 435-2372

Skype: Sales.atengineering

Web: www.ATengineering.us

REVISIONS				
Rev	Description	ECO	DATE	BY
1.0	Initial Release		12/10/11	Marty, Atec Inc.