



Ultra Light™ 1 Cheek Switch


REVISIONS			
Rev	Description	DATE	BY
1.0	Initial Release	3/6/14	MS, Atec Inc.

Introduction

The Ultra Light Cheek Switch is perfect for activating your favorite device with a simple cheek or chin movement.

Getting Started

- 1) Place the adjustable loop around the neck as shown in the photo to the right.
- 2) Adjust the loop so the end of the switch's lever is where you want it on the cheek or jaw.

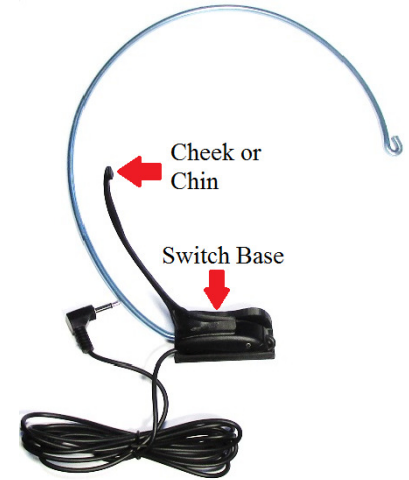

Note: In most cases you'll want the switch base directly below the chin.

Specifications

Activation Surface Area (in/cm)	0.16" x 0.16" (4mm x 4 mm)
Activation Force (oz/g)*	< 0.25 oz / < 7.09 g
Travel before Activation (in/cm)*	¼ inch (6.35mm)
Electrical Rating:	AC 125V 1A
Cable:	6 foot
Connector:	1/8 inch (3.5mm) Right angle mono plug standard for use with PC access, AAC, ECU, adapted Toys, etc.

Please don't hesitate to contact us if you have any questions.

Best Regards,

Assistive Technology Engineering and Consultation Inc. (Atec Inc.)

Sales: +1 (651) 315-7776

Fax: +1 (612) 435-2372

Email: Sales@ATengineering.us

Skype: Sales.atengineering

Web: www.ATengineering.us